










CRP Control

REF 343 001 1 x 1 ml

SYMBOLS IN PRODUCT LABELLING

| | | | |
|---|------------------------------|---|---------------------------------------|
|  | Authorised Representative |  | Temperature Limitation |
|  | For in-vitro diagnostic use |  | Use by/Expiration Date |
|  | Batch Code/Lot number |  | CAUTION. Consult instructions for use |
|  | Catalogue Number |  | Manufactured by |
|  | Consult instructions for use | | |

Intended Use

Accuracy control for the determination of C- Reactive proteins in serum by turbidimetry and nephelometry.

Composition

A dilution of human plasma and pleural fluid containing high levels of CRP with phosphate buffered saline.

Liquid stabilised.

Preservative: 0.095 g% sodium azide .

Storage and Stability

The expiry date of the product at +2 to +8 °C is listed on the label.

After first opening the control can be used for 6 weeks if stored tightly closed at +2 to+8 °C after use.

Do not freeze.

Precautions and Warnings

1.For "In Vitro Diagnostic Use"

2.Each individual donation intended for use in manufacturing of protein control serum was tested for hepatitis B surface antigen (HBsAg),anti-hepatitis C virus (anti-HCV) and anti-HIV1 and HIV2 by FDA required test. Only donations with negative findings were used for its manufacture.Nevertheless every product obtained from human body fluids should be handled with appropriate care in accordance with recommended procedures for biohazardous materials since absence of infectious agents can never be proven .

3.Reagents containing sodium azide must be handled with due caution: Do not ingest or allow to contact skin or mucous membranes !

Assigned Value

The value is lot dependent. Actual value is printed on the label.



Egyptian Company for Biotechnology (S.A.E)

Obour city industrial area. block 20008 piece 19 A. Cairo. Egypt.

Tel: +202 4665 1848 - Fax: +202 4665 1847

www.spectrum-diagnostics.com

E-mail:info@spectrum-diagnostics.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



IFUFTI44

Rev.(4), 30/11/2014

