










Procalcitonin calibrator set

REF: 567 001-CS (5 x 1 ml)

SYMBOLS IN PRODUCT LABELLING

	Authorised Representative		Temperature Limitation
	For in-vitro diagnostic use		Use by/Expiration Date
	Batch Code/Lot number		CAUTION. Consult instructions for use
	Catalogue Number		Manufactured by
	Consult instructions for use		

Intended Use

The Procalcitonin (PCT) Immunoassay Calibrator kit is used for the calibration of the quantitative PCT Immunoassay .

Characteristics

The PCT Immunoassay Calibrator kit is freeze-dry powder form.It contains one set of five levels (each 1.0 mL) of PCT provided in a human serum base.actual value of each calibrator is printed on vial label.

Storage and Stability

The PCT Immunoassay calibrators are stable up to expiration date when stored at 4 °C

Preparation

Reconstitute each vial with 1.0 mL bi distilled water ,allow to stand at room temperature for 15 minutes and mix gently by swirling until thoroughly dissolved. The reconstituted controls are stable for 5 days at 2 - 8 °C,and they can be stored in aliquots at - 20°C for 6 months from reconstitution.

Precautions and Warnings

- 1.For "In Vitro Diagnosticc Use"
- 2.Each donor unit of serum in the preparation of this material was tested by FDA-approved methods and found negative for the Human Immuno- deficiency Virus Antibody (HIV I/II Ab), Hepatitis B Surface Antigen(HBsAg), and Hepatitis C Virus Antibody (HCV). Because no method can offercomplete assurance as to the absence of infectious agents,however, this material and all patient samples should be handled as though capable of transmitting infectious disease and disposed of ac- cordingly.
- 3.Avoid contact with skin and eyes.It is harmful if swallowed,keep container tightly closed and keep away from food, drink, and animal feed.
- 4.Contains sodium azide, which may react with lead or copper plumbing to form explosive compounds. Flush drains with copious amounts of water when disposing of the material.

Calibrator Information

The Concentration of calibrator is stated on the vial label.



Egyptian Company for Biotechnology (S.A.E)
 Obour city industrial area. block 20008 piece 19 A. Cairo. Egypt.
 Tel: +202 4665 1848 - Fax: +202 4665 1847
 www.spectrum-diagnostics.com
 E-mail:info@spectrum-diagnostics.com



MDSS GmbH
 Schiffgraben 41
 30175 Hannover, Germany



